

**COMPUTERIZED TONOMETER**

# **CT-80A/80** *NEW*



# Topcon's new line-up for non-contact tonometry : CT-80A/80

— It's Topcon's solution for the further pursuit of operational use and speedy measurement, relieving patient's discomfort to its minimum.

## Fast and Easy Measurements


### The 3D Auto Alignment (CT-80A only)

The 3D Auto Alignment System offers an easy operation and speedy measurement even for patients with difficult fixation.



### 30% Faster Interval Time

The CT-80A/80 provide easier and faster operation with the aid of a new alignment bar and an improved air cylinder. The interval between measurements has been reduced by over 30%, so patients may be examined more comfortably in a shorter period of time. Once the alignment has been completed, the resulting data are automatically measured and displayed.

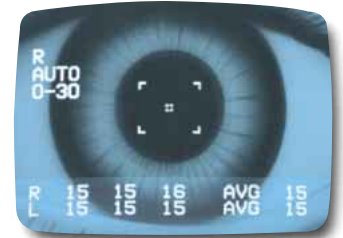

### Flex Control Function (CT-80A only)

With auto-alignment on, you can pick the perfect timing for the measurement, minimizing patient's discomfort. This is very suitable for difficult patients with moving eye and frequent blinking.



Keep tracking while holding the button.

Release the button, and measurement is done.

COMPUTERIZED TONOMETER

**CT-80A/80**


**NEW**



## A Focus on Safety

### Triple Safety Function

The CT-80A/80 features an innovative triple safety function. First, the operator may set an adjustable safety stopper. Second, a "too close" display on the monitor guides the user in alignment. Third, an audible warning buzzer informs the operator if the cornea and the instrument are too close.

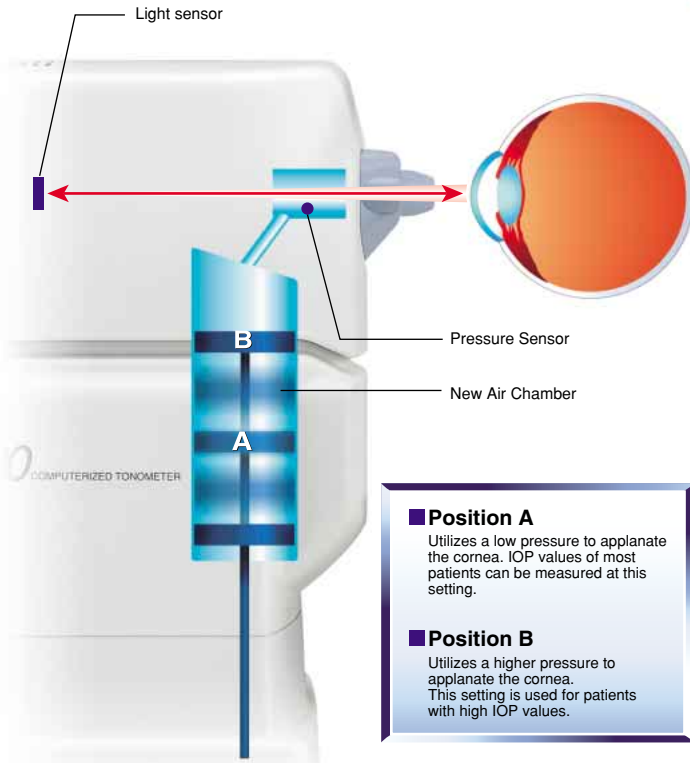


## Accurate Measuring System

### Dual Sensor System

The TOPCON CT-80A/80 incorporates an original measurement method using two sensors; one for light and one for pressure. When the air puff applanates the cornea, a light sensor detects the applanation moment, while at the same time another sensor monitors the internal chamber to obtain the intraocular pressure.

This new calculating system allows the operator to obtain highly precise and accurate measurement data without influence of atmospheric pressure, altitude or any possible mechanical degradation.


Measurement Data

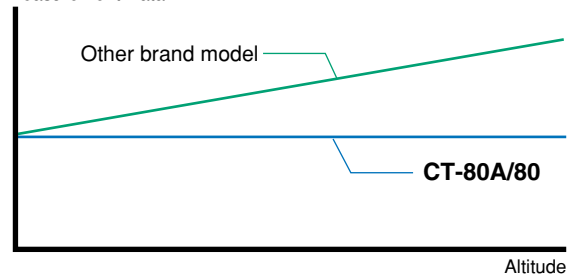

## Operating Conveniences

### Built-in Self Check Function

The CT-80A/80 lets the operator know in advance when maintenance is needed. It displays a warning message on the monitor when a lens is dirty, and also which lens it is (the measurement glass or the chamber glass).

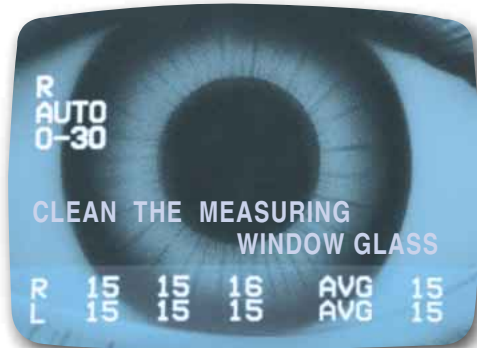

### Speedy Built-in Printer

At a touch of a button, the CT-80A/80 provides high-speed printout of measured data in about a couple of seconds! Up to three individual measurements plus an averaged value will be printed out.

NAME				
26/JAN/2001 PM 05:22				
		mmHg		AVG
R	16	17	17	17
L	16	16	17	16
CT-80 TOPCON				

### Optional Manual Control

The CT-80A/80 can be operated in a fully manual mode by overriding the automatic measurement setting.

### Internal Fixation Target

Infrared lighting provides the main illumination for the CT-80A/80: eliminating the need for harsh, bright alignment lighting and increasing patient comfort.

## ● Specifications

	CT-80A	CT-80
Measurement Range	0 –60mmHg [0 –30mmHg / 0 –60mmHg(Selectable)]	
Working Distance	11mm	
Measurement Display	TV Monitor Screen *R/L change-over . . . automatically detected and displayed. *Up to three measurements of each eye can be displayed and printed out.	
Measurement Recording	Built-in printer	
Measurement Mode	Auto Start with 3D Auto Alignment or Manual (selectable)	Auto Start or Manual (selectable)
Error Indication	When the measurement signal is not strong enough, IOP value is indicated in ( ) or just 'ERR' is displayed.	
Adjustable Safety Stopper	Minimum distance between the front of the instrument and the patient can be established in advance.	
Operating Temperature	10℃ to 40℃	
Base Travel	44mm (back & forth) / 88mm(left & right) / 28mm(vertically)	
Chinrest Vertical Travel	68mm	
External Output Terminal	RS-232C	
Power Supply	AC100-120V, 220-240V 50/60Hz	
Power Consumption	80VA ("Auto-shut-off" : power saving mechanism adopted)	
Dimensions	272(W) ×505(D) ×430 ~458(H)mm	
Weight	21kg	18kg

\*Subject to change in design and/or specifications without advanced notice.



TOPCON CORPORATION



Certificate No.Q18661  
TOPCON CORPORATION



Certificate No.EC97J1081  
TOPCON CORPORATION

**IMPORTANT**

In order to obtain the best results with this instrument,  
please be sure to review all user instructions prior to operation.



TOPCON CORPORATION

75-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8580, Japan. Phone: 3-3558-2520 Fax: 3-3960-4214 www.topcon.co.jp

### TOPCON AMERICA CORPORATION

CORPORATE OFFICE  
37 West Century Road, Paramus, NJ 07652, U.S.A.  
Phone: 201-261-9450 Fax: 201-387-2710 www.topcon.com

### TOPCON OMNI SYSTEMS, INC.

Valley Forge Business Center, 2430 Blvd. of the Generals,  
Norristown, PA 19403, U.S.A.  
Phone: 610-630-9200 Fax: 610-630-6428 www.topconomni.com

### TOPCON CANADA INC.

110 Provencher Avenue, Boisbriand QC J7G 1N1, CANADA  
Phone: 514-430-7771 Fax: 514-430-6457 www.topcon.ca

### TOPCON EUROPE B.V.

Esse Baan 11, 2908 LJ Capelle a/d IJssel, THE NETHERLANDS  
Phone: 010-4585077 Fax: 010-4585045 www.topconeuropa.com

### TOPCON S.A.R.L.

HEAD OFFICE  
89, Rue de Paris 92585 Clichy, Cedex, FRANCE  
Phone: 01-41069494 Fax: 01-47390251

### LYON OFFICE

138 Avenue du 8 Mai 1945, 69100 Villeurbanne, FRANCE  
Phone: 0478688237 Fax: 0478681902

### TOPCON DEUTSCHLAND G.m.b.H.

Weidkamp 180 45356 Essen, GERMANY  
Phone: 201-8619-200 Fax: 201-8619-275 www.topcon.de

### TOPCON SCANDINAVIA A.B.

Neogatan 2 S-43151 Molndal, SWEDEN  
Phone: 31-7109200 Fax: 31-7109249

### TOPCON ESPAÑA S.A.

HEAD OFFICE  
Frederic Mompou 5, 08960 Sant Just Desvern Barcelona, SPAIN  
Phone: 34-93-4734057 Fax: 34-93-4733932 www.topconesp.com

### MADRID OFFICE

Avenida Burgos, 16E, 1, 28036, Madrid, SPAIN  
Phone: 34-91-302-4129 Fax: 34-91-383-3890

### TOPCON (GREAT BRITAIN) LTD.

Topcon House, Kennet Side, Bone Lane, Newbury,  
Berkshire RG14 5PX United Kingdom  
Phone: 01635-551120 Fax: 01635-551170

### TOPCON SINGAPORE PTE.LTD.

Alexandra Distripark, Block 4, #05-15, Pasir Panjang Road, SINGAPORE 118491  
Phone: 2780222 Fax: 2733540 www.topcon.com.sg

### TOPCON AUSTRALIA PTY.LTD.

408 Victoria Road, Gladesville, NSW 2111, AUSTRALIA  
Phone: 02-9817-4666 Fax: 02-9817-4654 www.topcon.com.au

### TOPCON INSTRUMENTS (MALAYSIA) SDN. BHD.

Lot 226 Jalan Negara 2, Pusat Bandar Taman Melawati, Taman Melawati,  
53100 Kuala Lumpur, MALAYSIA  
Phone: 03-41079801 Fax: 03-41079796

### TOPCON INSTRUMENTS (THAILAND) CO.,LTD.

77/162 Sinn Sathorn Tower, 37th Floor, Krungthoburi Road,  
Klongtongnai, Klongsarn, Bangkok, 10600 THAILAND  
Phone: 662-440-1152-7 Fax: 662-440-1158

### TOPCON KOREA CORPORATION

Hyokong Bldg., 1306-1, Seocho-Dong, Seocho-Gu, Seoul, KOREA  
Phone: 02-3482-9231 Fax: 02-3481-1928 www.topcon.co.kr

### TOPCON OPTICAL(H.K.) LTD.

2/ F., Meeco Industrial Bldg., No.53-55 Au Pui Wan Street,  
Fo Tan Road, Shatin, N.T., Hong Kong  
Phone: 2690-1328 Fax: 2690-2221 www.topcon.com.hk

### TOPCON CORPORATION BEIJING OFFICE

Room NO.962 Poly Plaza Building, 14 Dongzhimen  
Nandajie, Dongcheng District, Beijing, 100027 CHINA  
Phone: 10-6501-4191-2 Fax: 10-6501-4190

### TOPCON CORPORATION SHANGHAI OFFICE

Room 901-903 Lippo Plaza No.222 Huai Hai Zhong Lu Shanghai, 20021 CHINA  
Phone: 21-62701877 Fax: 21-53966485

### TOPCON CORPORATION BEIRUT OFFICE

P.O. BOX 70-1002 Antelias, BEIRUT-LEBANON  
Phone: 961-4-523525/523526 Fax: 961-4-521119 E-mail: topconlb@destination.com.lb

### TOPCON CORPORATION DUBAI OFFICE

Office No.102,KHALAF FASHD AL MAILY BLDG., 245 Abu Hail Road, Deira, Dubai, UAE  
Phone: 971-4-2696511 Fax: 971-4-2696272 E-mail: topconme@emirates.net.ae